


Instructions for updating the HRV-Scanner software


The HRV-Scanner software provides two ways to update the software:

Automatic online Update

To run the automatic update, the computer on which the HRV-Scanner software is installed must have a Internet connection. Enable the automatic update in the System Settings.

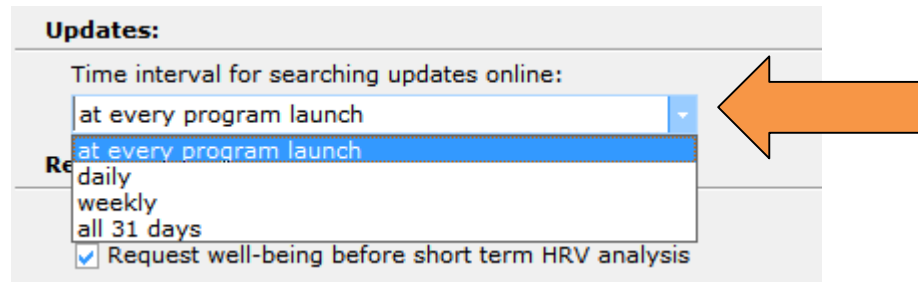


Specify the time interval for searching for new updates in the Internet, or start the online update manually (System -> Install Updates).

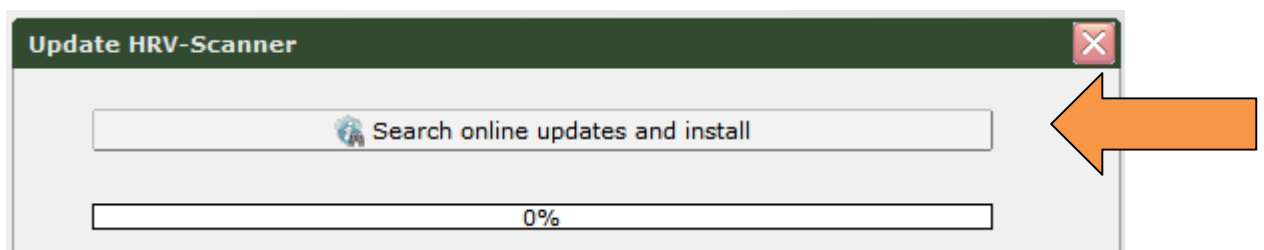

Follow the steps for the online update when a new update is found on the Internet.

Install Update manually from an Update File

Download the latest update file from our website. Please go to the BioSign website (www.biosign.de). Under "HRV-Scanner - Downloads" you will find the link to the current update file. Download this file and save it on a data carrier (such as a USB flash drive) if you want to install the update on a computer that does not have its own internet connection.

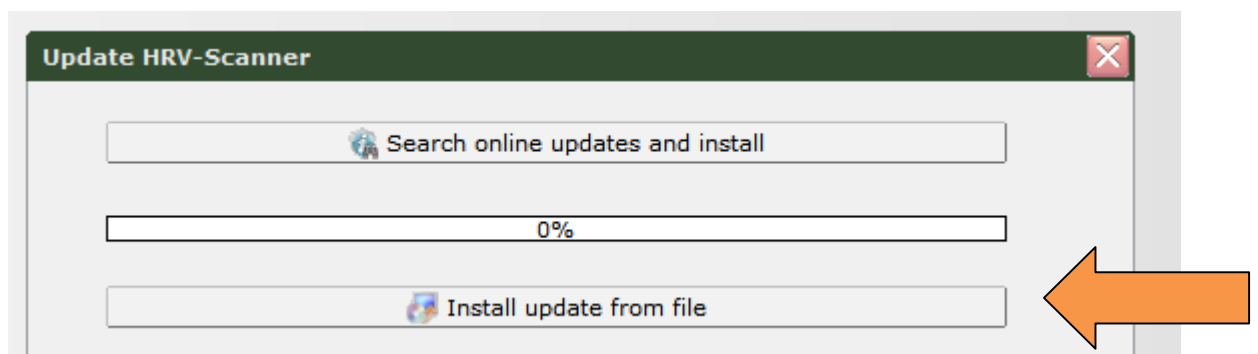

Go to "Install Updates" and select "Install update from file". Select the file downloaded from the Internet and install the update. (Note: The update file is a ZIP archive and can't be extracted.)

If you have any questions about the update or problems, please do not hesitate to contact us:

info@biosign.de