


## Installation of the Qiu-Driver

The Qiu is connected to a PC via USB and therefore needs a suitable driver. Sometimes restricted user rights or an anti virus-software don't allow that the Qiu-Driver is installed automatically. In this case a manual installation of the Qiu driver is required.

1. Exit the Qiu-software and disconnect the USB connection to the Qiu.
2. You can download the latest driver here:  
[http://www.biosign.de/download\\_Qiu/QiuDriverSetup.exe](http://www.biosign.de/download_Qiu/QiuDriverSetup.exe)
3. Please bookmark the folder where the driver file is located.
4. After the download is finished start the windows-explorer and navigate into the folder where the driver file is located
5. Right-click on the driver file and run the installation as an administrator.

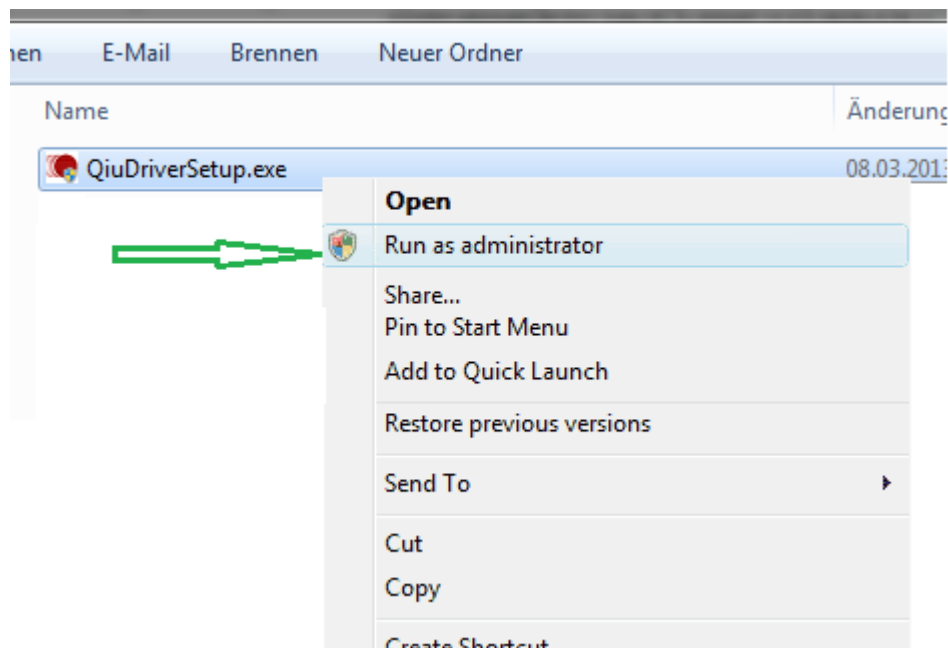

6. Please restart your PC once the installation is finished.
7. Now connect the Qiu to your PC. The new Qiu-Driver then will be installed. This may take several minutes. Please do not interrupt this process.
8. Now start the Qiu-Software. The Qiu should now be properly recognized by the software.

If there still are any connection problems, please do not bother to contact us to receive further technical support.